

Fitting and Care

LacelT™ LSO Products

(301 Corset LSO, 401 LSO, X641 LSO, X650 LSO)

Place handles as close to the closing system as possible. Be sure handles are next to the closing system. Stand holding product side panels parallel to floor. Pull outward to fully extend cord.

Position the LSO around the body, making sure the closing system is centered on the spine. Supports holding the closing system must be equally spaced on each side of the spine. Adjust up or down along the spine to determine the best support location.

Once location is determined, pull slightly outward on the side panels of the LSO to maximize an open position before using the fabric fasteners to close the front, right over left.

User Tips

- Pull handles only slightly at first until you get used to the power.
- Tighten to stand, loosen to sit. User controls the tension.
- Pulling in stomach and muscles before tightening makes pulling handles easier.
- Do not wear tight for long periods of time unless instructed by physician.
- The product was designed to enable adjustment throughout the day to provide more or less support as needed.
- To disconnect closing system, simply pull closing system away from fabric fastener. Attach closing system by placing closing system on fabric fastener.

Care Instructions

Remove closing system from side panels. Hand wash in cool water with mild soap. Thoroughly rinse and blot excess water from support with a clean towel. Air dry away from the sun.

Attachments

LacelT™ Solid Posterior Frame

Lift up the elastic straps located on the inside posterior portion of the LSO. Position the LSO around the outside portion of the Solid Posterior Frame, making sure the LSO superior edge is facing up.

Align the LSO elastic straps over the Solid Posterior Frame alignment tabs, and secure the LSO by sliding the elastic straps through the Solid Posterior Frame tabs.

LacelT™ Anterior Panel

Locate the anterior panel pocket found on the inside lining of the left side panel.

Lift up the anterior panel pocket and insert the rigid Anterior Panel.

LacelT™ Lateral Contour Pads

(Set of Two)

Removable contour pads may be positioned laterally along the inside side panel fabric fastener positioning strap. One or two pads may be used to accommodate user comfort.

LacelT™ Lordotic Pads

(Set of Two)

Lift up the elastic straps located on the inside posterior portion of the LSO. Place the Lordotic Pads under the elastic straps to secure proper placement.

LacelT™ Reusable Cold/Hot Therapy Gel Pack

Lift up the elastic straps located on the inside posterior portion of the LSO. Place the reusable Cold/Hot Therapy Pack under the elastic straps to secure proper placement. Follow the safety instructions located on the back of each gel pack.

LacelT™ X640 Dynamically Self-Adjusting Frame™

Unscrew Secure Slide™ fastener from post located on posterior side of the Dynamically Self-Adjusting Frame™.

Place the LSO around the outside circumference of the Dynamically Self-Adjusting Frame™, making sure LSO superior edge is facing up.

Align LSO making sure posterior panels Secure Slide™ is centered over Secure Slide™ post located on the Dynamically Self-Adjusting Frame™

To fasten LSO to frame, simply attach Secure Slide™ fastener to post.
