

### **LaceIT™ Anterior Panel**

Locate the anterior panel pocket found on the inside lining of the left side panel.

Lift up the anterior panel pocket and insert the rigid Anterior Panel.

### **LaceIT™ Lateral Contour Pads**

(Set of Two)

Removable contour pads may be positioned laterally along the inside side panel fabric fastener positioning strap. One or two pads may be used to accommodate user comfort.

### **LaceIT™ Lordotic Pads**

(Set of Two)

Lift up the elastic straps located on the inside posterior portion of the LSO. Place the Lordotic Pads under the elastic straps to secure proper placement.

### **LaceIT™ Reusable Cold/Hot Therapy Gel Pack**

Lift up the elastic straps located on the inside posterior portion of the LSO. Place the reusable Cold/Hot Therapy Pack under the elastic straps to secure proper placement. Follow the safety instructions located on the back of each gel pack.

### **LaceIT™ X640 Dynamically Self-Adjusting Frame™**

Unscrew Secure Slide™ fastener from post located on posterior side of the Dynamically Self-Adjusting Frame™.

Place the LSO around the outside circumference of the Dynamically Self-Adjusting Frame™, making sure LSO superior edge is facing up.

Align LSO making sure posterior panels Secure Slide™ is centered over Secure Slide™ post located on the Dynamically Self-Adjusting Frame™

To fasten LSO to frame, simply attach Secure Slide™ fastener to post.

---